**ACE CAL****Enzima Convertidora de Angiotensina /  
Angiotensin Converting Enzyme  
Calibrador – Calibrator****Angiotensin Converting Enzyme (ACE)  
Calibrator  
IVD**  
Store at 2 - 8°C**PRODUCT CHARACTERISTICS**

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**STORAGE AND STABILITY**

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth. Store calibrators tightly capped when not used. Discard the CONTROL in case of evident microbial contamination.  
After reconstitution is stable for:  
At 2°C to 8°C 7 days

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ACE CAL is a human lyophilised serum.  
ACE Calibrator is intended to be used with ACE Liquid Reagent for the preparation of the calibration curve for the kinetic determination of angiotensin converting enzyme (ACE) assay in human serum or plasma.

**PRECAUTIONS**

Human based serum from selected donors was used in the manufacturing of this product. Each donor unit was tested with FDA licensed reagents and found non-reactive for HBsAg, HCV, HIV1 and HIV2 antibodies. Because no test method can offer complete assurance that human blood derived products will not transmit infectious agents, it is recommended that this product be handled with the same precautions used for patient specimens.

**PREPARATION**

-Reconstitute (→) with 1,0 mL of distilled water.  
-Dissolve contents completely within 5 to 10 min. by swirling gently, avoiding foam forming. Do not shake.  
We recommend preparing the sera protected from light.

**STORAGE AND STABILITY**

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth. Store calibrators tightly capped when not use. Discard the CALIBRATOR in case of evident microbial contamination.  
After reconstitution is stable for:  
At 2°C to 8°C 7 days

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ACE CAL is a human lyophilised serum.  
ACE Calibrator is intended to be used with ACE Liquid Reagent for the preparation of the calibration curve for the kinetic determination of angiotensin converting enzyme (ACE) assay in human serum or plasma.

**PRECAUTIONS**

Human based serum from selected donors was used in the manufacturing of this product. Each donor unit was tested with FDA licensed reagents and found non-reactive for HBsAg, HCV, HIV1 and HIV2 antibodies. Because no test method can offer complete assurance that human blood derived products will not transmit infectious agents, it is recommended that this product be handled with the same precautions used for patient specimens

**PREPARATION**

-Reconstitute (→) with 1,0 mL of distilled water.  
-Dissolve contents completely within 5 to 10 min. by swirling gently, avoiding foam forming. Do not shake.  
We recommend preparing the sera protected from light.

**STORAGE AND STABILITY**

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth. Store calibrators tightly capped when not use. Discard the CALIBRATOR in case of evident microbial contamination.  
After reconstitution is stable for:  
At 2°C to 8°C 7 days

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