



**NH3/EtOH Control serum
IVD**

Store at 2 - 8°C

PRODUCT CHARACTERISTICS

The Ammonia Ethanol control is a human liquid control serum. Is intended for quality control procedures in examining the accuracy and precision of quantitative Ammonia and Alcohol assays.

For *in vitro* diagnostic use only.

PRECAUTIONS

Each donor unit in the preparation of this material was tested by FDA-approved method and found negative for Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody. However this material and all patient samples should be handled as if potentially infectious and disposed accordingly.

PREPARATION

Ready for use.

STORAGE AND STABILITY

The Control serum is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

Once opened, the product is stable for 30 days at 2-8°C. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

PACKAGING

REF 1002240 Cont. 4 x 2 mL

**Suero control NH3/EtOH
IVD**

Conservar a 2 - 8°C

CARACTERÍSTICAS DEL PRODUCTO

El control de Amoniaco Etanol es un suero humano líquido. Está destinado al control de calidad de la exactitud y precisión del procedimiento de determinación cuantitativa de Amoniaco y alcohol.

Sólo para uso de diagnóstico *in vitro*.

PRECAUCIONES

Cada unidad de donante utilizada para la preparación de este material ha resultado ser negativa, en test aprobados por la FDA, para anticuerpo del Virus de Inmunodeficiencia Humana (HIV 1, HIV 2), HbsAg y anticuerpo del Virus de Hepatitis C (HCV). Sin embargo, deben manipularse y desecharse con precaución como potencialmente infecciosos.

PREPARACIÓN

Listo para su uso.

CONSERVACIÓN Y ESTABILIDAD

El Suero control es estable hasta la fecha de caducidad indicada en el envase cuando se mantiene el vial bien cerrado a 2-8°C, y se evita la contaminación durante su uso.

Una vez abierto, el producto es estable durante 30 días cuando se mantiene a 2-8°C. Sólo se debe sacar la cantidad a utilizar. Después de su uso, no se debe devolver al vial original ningún producto residual.

PRESENTACIÓN

REF 1002240 Cont. 4 x 2 mL

LOT [] [] []

ASSIGENED VALUES / VALORES ASIGNADOS

Analyte / Analito	Target / Valor	Expected Range / Intervalo
Ammonia / Amoniaco (µmol/L)	148	118 - 178
Ethanol / Etanol (g/L)	1.60	1.28 – 1.92

*This Values sheet is applicable to lot and sublots. Sequential alphabetical letter (e.g. A, B, C etc.) following the lot number.
*Esta hoja de valores es aplicable al lote y sublotes. Los sublotes se indican con el no. lote seguido de letras secuenciales.