



tHcy CONTROL

HOMOCISTEÍNA / HOMOCYSTEIN
Control - Control

**tHcy Control serum
IVD**

Store at 2 - 8°C

PRODUCT CHARACTERISTICS

The Homocysteine Control Kit is used as control serum for quantitative measurement of total L-homocysteine in human serum or plasma and for quality control procedures in examining the accuracy and precision of quantitative Homocystein assays.

REAGENTS

Homocystein controls are a lyophilized material prepared with human serum base.

The concentrations of the tHcy are lot-specific. See chart below for the values.

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

PREPARATION

Ready for use. No control preparation is required.

STORAGE AND STABILITY

Controls are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

Do not use reagents over the expiration date.

If there is evidence of microbial contamination or excessive turbidity in the product discard the vial.

This product is not intended for use as a calibrator.

PACKAGING

REF 1002281 Cont. 3 x 1,5 mL

PRESENTACIÓN

REF 1002281 Cont. 3 x 1,5 mL

LOT [] []

ASSIGNED VALUES / VALORES ASIGNADOS

LEVEL / NIVEL	Target / Valor	Expected Range / Intervalo
1	µmol/L	µmol/L
2	µmol/L	µmol/L
3	µmol/L	µmol/L

*This Values sheet is applicable to lot and sublots. Sequential alphabetical letter (e.g. A, B, C etc.) following the lot number.
*Esta hoja de valores es aplicable al lote y sublotes. Los sublotes se indican con el no. lote seguido de letras secuenciales.

