BZO CTL 200 Urine

Benzodiazepines / Benzodiazepina

Low Cut-off Urine Control 200 / Control Cut-off Bajo Orina 200

DoA Liquid Test

Benzodiazepine Control Low Cut-off (200 ng/mL) for Urine IVD

Store at 2 - 8°C.

INTENDED USE

The Benzodiazepine Control Low Cutoff is consisting of two levels $\pm 25\%$ from the low cutoff (200ng/mL) for the control of Spinreact DoA-EIA liquid test for Benzodiazepines.

REAGENTS

Level - 25%	Benzodiazepine Control Low Cut-off (200 ng/mL) Human Urine. Sodium azide 0.5 g/L Oxazepam
Level + 25%	Benzodiazepine Control Low Cut-off (200 ng/mL) Human Urine. Sodium azide 0.5 g/L Oxazepam

Each level has the following concentrations:

ng/mL	- 25 %	+ 25 %
Oxazepam	150	250

PREPARATION AND STABILITY

The controls are ready to use. No control preparation is required. The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use.

WARNING AND PRECAUTIONS

- The controls are for in vitro diagnostic use only. Harmful if swallowed.
- The controls contain sodium azide that may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such controls or wastes always flush with a large volume of water to prevent azide build-up.
- Do not use the controls beyond their expiration dates.
- Keep all containers closed when not in use to avoid microbial contamination.
- Do not mix controls from different manufacturers.
- Do not freeze controls.

Handle all urine specimens as if they were potentially infectious.

PROCEDURE

The controls should be used in the Benzodiazepine reagent for urine.

REFERENCES

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Spinreact Benzodiazepines EIA (Refs #930018/930020) package inserts.

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REFERENCIAS

1. Urine testing for Drug of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. European Laboratory Guidelines for Legally Defensible Workplace Drug Testing – Urine Drug Testing, EWDTS, 2002.
3. Hoja de instrucciones del kit Spinreact Benzodiazepina-EIA (Refs # 930018/930020).

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Ref. 936081 **CONT.** Nivel -25 %: 2x10 mL
Nivel +25%: 2x10 mL