## Quantitative determination of albumin IVD

Store at 2-8°C

### PRINCIPLE OF THE METHOD

Albumin in the presence of bromcresol green at a slightly acid pH, produces a colour change of the indicator from yellow-green to green-blue. The intensity of the color formed is proportional to the albumin concentration in the sample<sup>1,2,3,4</sup>.

### CLINICAL SIGNIFICANCE

One of the most important serum proteins produced in the liver is albumin.

This molecule has an extraordinarily wide range of functions, including nutrition, maintenance of oncotic pressure and transport of Ca<sup>++</sup>, bilirubin, free fatty acid, drugs and steroids.

Variation in albumin levels indicate liver diseases, malnutrition, skin lesions such as dermatitis and burns or dehydration<sup>1,7,8</sup>.

Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

### REAGENTS

R	Bromcresol green pH 4,2	0,12 mmol/L
---	-------------------------	-------------

### PREPARATION

The reagent is ready to use.

### STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use.

Do not use reagents over the expiration date.

### Signs of reagent deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 600 nm  $\geq$  0,40.

### ADDITIONAL EQUIPMENT

- SPIN640 / SPIN640Plus Autoanalyzer.
- General laboratory equipment.

### SAMPLES

Serum or plasma, free of hemolysis<sup>1</sup>: Stability 1 month at 2-8°C or 1 week at 15-25°C.

### REFERENCE VALUES

3,5 to 5,0 g/dL<sup>1</sup>.

These values are for orientation purpose; each laboratory should establish its own reference range.

### QUALITY CONTROL

Control sera and calibrators are recommended to monitor the performance of assay procedures: SPINROL H Calibrator, SPINROL H Normal and Pathologic (Ref. 1002011, 1002120 and 1002210).

If control values are found outside the defined range, check the instrument, reagents and technique for problems.

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

**BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.**

## SPIN640 APPLICATION

TEST INFORMATION		REAGENT VOLUME			
Nº	**	Vol. R1	350		
Test	ALB	Vol. R2			
Full Name	Albumin	Vol. R3			
Standard nº	1	Vol. R4			
SAMPLE VOLUME		RESULT SETUP			
Vol. Sample Stand.	2	Decimal	0.01	Slope	1
Vol. Sample Increas.		Unit	g/dL	Inter.	0
Vol. Sample Dec					
REACTION PARAMETERS					
Reac. Type	End point	Direction	Increase		
Pri. Wave.	605	Reagent Blank	10-11		
Sec. Wave.		React. Time	79-80		

## SPIN640Plus APPLICATION

EDIT PARAMETERS			
Test	ALB	No.	**
Full name	ALBUMIN	Print name	ALB
Reac. Type	End Point	Direction	Increase
Pri. Wave.	605	Sec. Wave.	
Unit	g/dL	Decimal	0.01
Reagent Blank	10 - 11	React. Time	79 - 80
Vol. Sample	1.5 ul	R1	300 ul
Increased		R2	
Decreased		R3	
Sample blank		R4	

*The Calibration is stable until **36 days**. After this period the Calibration must be performed again in order to obtain good results.*

## PERFORMANCE CHARACTERISTICS

**Measuring range:** From detection limit of 0,0349 g/dL to linearity limit of 6 g/dL.

If the results obtained were greater than linearity limit, dilute the sample 1/2 with NaCl 9 g/L and multiply the result by 2.

### Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
Mean (g/dL)	5,00	3,71	4,56	3,07
SD	0,02	0,02	0,28	0,18
CV (%)	0,47	0,55	6,20	5,90

**Sensitivity:** 1 g/dL = 0,2003 A.

**Accuracy:** Results obtained using SPINREACT reagents (y) did not show systematic differences when compared with other commercial reagents (x).

The results obtained using 50 samples were the following:

Correlation coefficient (r)<sup>2</sup>: 0,99169.

Regression equation: y= 1,045x - 0,028.

The results of the performance characteristics depend on the analyzer used.

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