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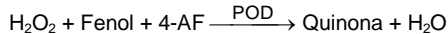
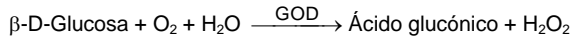
1. Kaplan L.A. Glucose. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1032-1036.
2. Trinder P. Ann Clin Biochem 1969; 6: 24-33.
3. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
4. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
5. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
6. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

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Ref: M141011

 Cont.

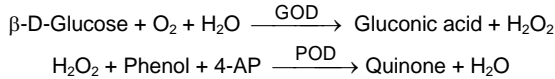
R: 6 x 30 mL

Quantitative determination of glucose IVD

Store at 2-8°C

PRINCIPLE OF THE METHOD

Glucose oxidase (GOD) catalyses the oxidation of glucose to gluconic acid. The formed hydrogen peroxide (H₂O₂), is detected by a chromogenic oxygen acceptor, phenol, 4 – aminophenazone (4-AP) in the presence of peroxidase (POD):

The intensity of the color formed is proportional to the glucose concentration in the sample^{1,2}.

CLINICAL SIGNIFICANCE

Glucose is a major source of energy for most cells of the body; insulin facilitates glucose entry into the cells. Diabetes is a disease manifested by hyperglycemia; patients with diabetes demonstrate an inability to produce insulin^{1,5,6}. Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

REAGENTS

R	TRIS pH 7.4	92 mmol/L
	Phenol	0,3 mmol/L
	Glucose oxidase (GOD)	15000 U/L
	Peroxidase (POD)	1000 U/L
	4 – Aminophenazone (4-AP)	2,6 mmol/L

PREPARATION

The reagent is ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use. Do not use reagents over the expiration date.

SIGNS OF REAGENT DETERIORATION:

- Presence of particles and turbidity.
- Blank absorbance (A) at 505 nm ≥ 0,32.

ADDITIONAL EQUIPMENT

- MINDRAY BS-120 / BS-200E Autoanalyzer.
- General laboratory equipment.

SAMPLES

Serum or plasma, free of hemolysis¹:
 Serum should be removed from the clot as quickly as possible.
 Stability of the sample: Glucose in serum or plasma is stable at 2-8° for 3 days.

REFERENCE VALUES¹

Serum or plasma:
 60 – 110 mg/dL ≅ 3,33 – 6,10 mmol/L
 These values are for orientation purpose; each laboratory should establish its own reference range.

QUALITY CONTROL

Control sera and calibrators are recommended to monitor the performance of assay procedures: SPINTROL H Calibrator, SPINTROL H Normal and Pathologic (Ref. 1002011, 1002120 and 1002210).
 If control values are found outside the defined range, check the instrument, reagents and technique for problems.
 Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

NOTES

1. Calibration with the aqueous standard may cause a systematic error in automatic procedures. In these cases, it is recommended to use a serum Calibrator.
2. Use clean disposable pipette tips for its dispensation.

MINDRAY BS-120 / BS-200E APPLICATION

<u>PARAMETERS</u>			
Test	GLU / GLU	R1	300 / 300
Nº	**	R2	*
Full Name	GLU / GLU	Sample volume	3 / 3
Standard Nº		R1 Blank	
Reac. Type	Endpoint / Endpoint	Mixed Rgt Blank	
Pri. Wavelength	510 / 505	Linearity Range	0 mg/dL 500 mg/dL
Sec. Wavelength		Linearity Limit	*
Direction	Increase / Increase	Substrate Limit	*
Reac. Time	0_33 / 0_33	Factor	*
Incuba. Time		Prozone check	*
Units	mg/dL / mg/dL	q1	q2
Precision	Interger / Interger	q3	q4
		PC	Abs
<u>CALIBRATION (Cal + Rgt Blk)</u>			
Rule	One-point Linear / Two-point Linear		
Sensitivity	1 / 1		
Replicates	2 / 2		
Interval (days)	0 / 0		
Difference Limit			
SD			
Blank Response			
Error Limit			
Correlation Coefficient			

Blank parameter must be performed in order to get good results in CALIB screen from main menu. The blank calibration is stable until **35 days**. After this period the blank parameter must be performed again in order to validate the calibration.

PERFORMANCE CHARACTERISTICS

Measuring range: From detection limit 0,3709 mg/dL to linearity limit 500 mg/dL.

If the concentration is greater than linearity limit dilute 1/2 the sample with CINA 9 g/L and multiply the result by 2.

Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
Mean (mg/dL)	98,5	264,6	92,5	250
SD	0,5754	1,2733	2,76	6,44
CV (%)	0,59	0,48	2,98	2,57

Sensitivity: 1 mg/dL = 0,0039 (A).

Accuracy: Results obtained using SPINREACT reagents (y) did not show systematic differences when compared with other commercial reagent (x).

The results obtained using 50 samples were the following:

Correlation coefficient (r)²: 0,99492.

Regression equation: y=1,104x - 1,249.

The results of the performance characteristics depend on the analyzer used.

BIBLIOGRAPHY

1. Kaplan L.A. Glucose. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1032-1036.
2. Trinder P. Ann Clin Biochem 1969; 6 24-33.
3. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
4. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
5. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
6. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

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