
**Quantitative determination of aspartate aminotransferase
GOT (AST)
IVD**

Store at 2-8°C

PRINCIPLE OF THE METHOD

Aspartate aminotransferase (AST) formerly called glutamate oxaloacetate (GOT) catalyses the reversible transfer of an amino group from aspartate to α -ketoglutarate forming glutamate and oxalacetate. The oxalacetate produced is reduced to malate by malate dehydrogenase (MDH) and NADH: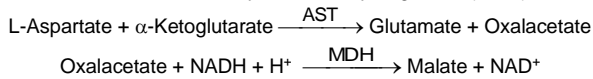

The rate of decrease in concentration of NADH, measured photometrically, is proportional to the catalytic concentration of AST present in the sample¹.

CLINICAL SIGNIFICANCE

The AST is a cellular enzyme, is found in highest concentration in heart muscle, the cells of the liver, the cells of the skeletal muscle and in smaller amounts in other weaves.

Although an elevated level of AST in the serum is not specific of the hepatic disease, is used mainly to diagnostic and to verify the course of this disease with other enzymes like ALT and ALP.

Also it is used to control the patients after myocardial infarction, in skeletal muscle disease and other^{1,4,5}.

Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

REAGENTS

R 1 Buffer	TRIS pH 7.8	80 mmol/L
	Lactate dehydrogenase (LDH)	800 U/L
	Malate dehydrogenase (MDH)	600 U/L
	L-Aspartate	200 mmol/L
R 2 Substrate	NADH	0,18 mmol/L
	α -Ketoglutarate	12 mmol/L

PRECAUTIONS

R1: H290-May be corrosive to metals.

Follow the precautionary statements given in MSDS and label of the product.

PREPARATION

All the reagents are ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use.

Do not use reagents over the expiration date.

Signs of reagent deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 340 nm < 1.00.

ADDITIONAL EQUIPMENT

- SPIN 800 Autoanalyzer.
- General laboratory equipment.

SAMPLES

Serum or plasma¹: Stability 7 days at 2-8°C.

QUALITY CONTROL

Control sera are recommended to monitor the performance of assay procedures: SPINROL H Normal and Pathologic (Ref. 1002120 and 1002210).

If control values are found outside the defined range, check the instrument, reagents and technique for problems.

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.

SPIN 800 APPLICATION

EDIT PARAMETERS			
Test	GOT	No.	**
Full name	GOT	Print name	GOT
Reac. Type	Kinetic	Direction	Decrease
Pri. Wave.	340	Sec. Wave.	
Unit	U/L	Decimal	0.1
Reagent Blank	-	React. Time	21 - 31
Vol. Sample	30 ul	Increas.	Decrease
			Sample blank
R1	240 ul	R2	60 ul
		R3	
		R4	
CALIBRATION			
Rule	Linear two points	Water	0
		Calibrator	*

The Calibration is stable until **35 days**. After this period the Calibration must be performed again in order to obtain good results.

REFERENCE VALUES¹

Men	38 U/L
Women	31 U/L

These values are for orientation purpose; each laboratory should establish its own reference range.

PERFORMANCE CHARACTERISTICS

Measuring range: From detection limit of 0 U/L to linearity limit of 467 U/L.

If the results obtained were greater than linearity limit, dilute the sample 1/10 with NaCl 9 g/L and multiply the result by 10.

Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
	Mean (U/L)	SD	Mean (U/L)	SD
Mean (U/L)	48,1	159	47,4	156
SD	0,56	0,57	1,42	4,35
CV (%)	1,16	0,36	3,00	2,79

Sensitivity: 1 U/L = 0,00053 ΔA /min.

Accuracy: Results obtained using SPINREACT reagents (y) did not show systematic differences when compared with other commercial reagents (x).

The results obtained using 50 samples were the following:

Correlation coefficient (r): 0,99956.

Regression equation: $y = 1,042x - 0,342$.

The results of the performance characteristics depend on the analyzer used.

BIBLIOGRAPHY

1. Murray R. Aspartate aminotransferase. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1112-1116.
2. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
3. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
4. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
5. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

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EDIT PARAMETERS			
Test	GOT	No.	**
Full name	GOT	Print name	GOT
Reac. Type	Kinetic	Direction	Decrease
Pri. Wave.	340	Sec. Wave.	
Unit	U/L	Decimal	0.1
Reagent Blank	-	React. Time	21 - 31
Vol. Sample	30 ul	Increas.	Decrease
			Sample blank
R1	240 ul	R2	60 ul
		R3	R4
CALIBRATION			
Rule	Linear two points	Water	0
		Calibrator	*

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- Murray R. Aspartate aminotransferase. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1112-1116.
- Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
- Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
- Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
- Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

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